



DUOGEL GEL CUSHION



Overview

Product Code: P311CD

Product Name: Duogel Gel Cushion

Description:

- Based on the multi-bearing surface concept, the Duogel cushion is made from two different viscoelastic gels.
- The white pad is a low bearing, viscoelastic, self-contained and leak-free polyurethane gel designed to reduce pressure on high-risk areas such as pelvic protuberances.
- The blue gel is a high bearing surface reticulated viscoelastic P.U gel for use on low risk areas to improve comfort and stability. The gel resists fluid migration, thus preventing penetration.

Dimensions: 420 x 420mm

Weight: 3kg

Max. User Weight: 22 Stone / 140kg

Pressure Relief: High Risk

Cover: Anti-allergenic, antibacterial, antimicrobial, anti-fungal, 2 way stretch, vapour permeable, washable, welded seams, 2 side zip and anti-slip base. Printed user instructions on cover.

Fire Retardancy: NF EN ISO 597 - 1 & 2

Cleaning Guidance:

The cushion foam may be hand washed with mild detergent. Do not immerse in water. Allow to dry naturally. Cover should be wiped down with hot water and mild detergent or machine washed up to 95°C and tumble dried. For a comprehensive list of tested cleaning products see our website.

Warranty

In terms of Harvest Healthcare Guarantee and Warranties the Duogel cushion is guaranteed for three years from date of purchase. Exclusions:

Exclusions:

1. Removable cushion covers are guaranteed for one year
2. Damage to fabric covers through incorrect use and penetration by sharp instruments is excluded

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk
Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Harvest Healthcare Ltd supplies a range of products such as foam mattresses, active air mattresses, fall protection mats and bed rail protection bumpers. These products are usually covered in vapour permeable PU fabric that could cause risk of suffocation in the event of prolonged facial contact.
(Mattress risk categories are to be used as a guide only)

Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

Version 2: Nov. 2017