

Intercall nurse call system connections.

HJR075 alarm system



The HJM301 FLOOR pressure pad can be plugged directly into the nurse call system (does not require the HJR075 alarm system) by using the HJM308 cable

HLV006 Y connector

HJM308 cable

HJM330.M Nurse call handset

HJM309 cable



HLV006 Y connector

1. Connect a HLV006 Y connector into nurse call system
2. Connect nurse call handset to one connector and a HJM309 cable to the other connector
3. Connect the other end of the HJM309 cable into the HJR075 alarm system socket marked NURSE CALL
4. Connect another HLV006 Y connector to the black cable already connected to the Alarm system.
5. Connect the HLV30M mattress pad to one connector and the HLV301S seat pad to the other.
6. If the chair is located away from the bed you can use a HLV006E 6-meter extension cable

HLV301M

Matress pad sensor



HLV301S

Seat pad sensor.

- Carry out a risk assessment to ensure this system is suitable for intended use and test system for correct operation each time before use.
- If using a seat pad and matress pad only one can be in use at any time.
- Read the HJR075 alarm system instructions carefully.
- If you have any doubts about the above please contact Harvest Healthcare LTD