

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
General User/ Safety Guide harvesthealthcare HARVEST STATIC

STATIC CUSHIONS





www.harvesthealthcare.co.uk



CONTENTS

WARNINGS & CAUTIONS	4
OVERVIEW	5
GENERAL INFORMATION	6-7
CLEANING & CARE	8
GUARANTEES & WARRANTIES	9
DECLARATION OF CONFORMITY	10

WARNINGS & CAUTIONS



READ THIS INSTRUCTION MANUAL CAREFULLY. PLEASE NOTE THE SAFETY INSTRUCTIONS MUST BE OBSERVED.

WARNING

Please refer to the information in this booklet as a guideline only.

Harvest Healthcare Ltd recommend that prior to using this product, a full risk assessment must be completed by a qualified professional, in order to determine that the correct product is suitable for each individual.

Do not use this product for/whilst lifting the patient. This will damage the product and could put the patient at risk.

Ensure that this product is used the correct way up.

Facial contact with the cover may cause suffocation.

Do not allow sharp objects to penetrate the cover material.

Do not expose this product to open flames. We advise the user not smoke when using this product.

SYMBOLS USED



WASH at

80°C





DO NOT

DRY CLEAN

DO NOT BLEACH



DO NOT IRON

TUMBLE DRY ON LOW

CE MARK



DO NOT USE PHENOLS



NO SMOKING



This symbol indicates general hazards. There is a danger to life and health.





CUSHION RANGE OVERVIEW

CUSHION	WIDTH / DEPTH (MM)	THICKNESS (MM)	MAX PATIENT WEIGHT (KG)	RISK LEVEL	BRADEN SCALE SCORE	WATERLOW SCORE
MODULAR FOAM HFC121	430	95	95	Medium	13 - 14	5 - 10
MODULAR FOAM HFC116 (LARGE)	500	95	108	Medium	13 - 14	5 - 10
VISCO SANDWICH HFC128	430	95	127	High	13 - 14	5 - 10
VISCO SANDWICH HFC128.1 (LARGE)	510	95	139	High	13 - 14	5 - 10
3-LAYER VISCO TOP HFC129.01	430	75	158	High	10 - 12	15 - 20
VISCO TOP HFC129	430	76	127	High	10 - 12	15 - 20
VISCO TOP HFC123 (LARGE)	510	76	139	High	10 - 12	15 - 20
REFLECT HFC130	430	100	127	High	10 - 12	15 - 20
REFLECT HFC132 (LARGE)	500	100	139	High	10 - 12	15 - 20

All Harvest Healthcare static cushion covers PU - stitched. They all have a Crib 5 fire rating.

GENERAL INFORMATION

INSTRUCTIONS FOR USE

Check that the safe working load of the cushion is suitable for the intended user. **Do not exceed the maximum patient weight of the cushion** (see page 5).

Ensure the Risk Factor is suitable for the intended user. To help prevent pressure damage it is important that patients reposition themselves, or are repositioned regularly.

Cushions are delivered in plastic covers for protection; these covers must be removed before use. Avoid using any additional covers/padding between the patient and the support surface, as this may affect its pressure reducing qualities.

Ensure that this product is used the correct way up and on a suitable chair.

PERFORMANCE CHARACTERISTICS/CUSHION INSPECTION RECOMMENDATIONS

Cushions should be checked regularly to ensure they remain 'fit for purpose', clinically effective and pose no risk of infection to either the patient or the carer.

Harvest Healthcare Ltd recommend that a thorough inspection of both the interior (foam core) and exterior (cover) of the cushion is carried out weekly, or each time a new patient is placed on the cushion. Visual checks should be carried out daily to identify any significant signs of damage or infection risk.

If any signs of contamination are identified on the cover, the mattress should be withdrawn from use immediately and the cover replaced. If the foam and/or the cover are showing signs of contamination or excessive wear the mattress should be removed from use immediately and the entire mattress replaced. Signs of excessive wear may include, but are not restricted to, thinning, discolouration, tearing, fraying, misshaping and fluid ingress.



Facial contact with any covering material such as that used on this product can cause suffocation and a Risk Assessment should be done prior to the use of this product.

CUSHION CLEANING AND CARE

Cushions are fully encased in a protective waterproof, multi-stretch, PU cover to prevent fluid ingress from damaging the foam core. The cushion cover should be cleaned regularly in accordance with your organisation's disinfection protocol.

For full cleaning procedures for our cushions see page 8.

GENERAL INFORMATION

MOVING, HANDLING AND STORAGE

Where possible, cushions should be stacked flat. Store cushions in a temperate, dry environment. Do not expose to high humidity or heat.

Do not drag or pull cushions by the cover.

CLEANING & CARE

Eye protection, gloves and protective clothing should be worn when carrying out cleaning and disinfection procedures.

When disinfecting the system, Harvest Healthcare Ltd recommends the following guidelines which have been developed to comply with recognised infection control procedures. These procedures are also to be used to prevent cross infection when transferring the system between patients.

CUSHION COVER

During general use the cushion can be cleaned by wiping with a mild detergent solution.

Where necessary the cushion cover can be removed for laundering or sterilisation. Where there is staining or body fluids on the cushion, wash thoroughly with soap and water, then wipe with a sodium hypochlorite solution diluted to 1000ppm before laundering.

Cushion covers may be laundered as follows:

- 1 Pre wash Cold 10 minutes
- 2 Main Wash 80°C 10 minutes
- **3** Followed by cold rinses and extraction.



Do not use abrasive cleaners, phenol disinfectants, solvents or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.



Do not iron.

Ensure the cushion is thoroughly dried before use or placing in storage.



HYPERCARBONATE AND PHENOL-BASED SOLUTIONS SHOULD NOT BE USED AS THESE WILL DAMAGE THE CUSHION COVER

DISPOSAL

If contaminated, please dispose of as clinical waste. If not contaminated dispose of as household waste at your local approved facility.

Note:

Please report any serious incident related to this device to Harvest Healthcare. Full contact details can be found at the end of this document.

GUARANTEES & WARRANTIES

WARRANTY

Foam Inner

All Harvest Healthcare static foam cushions are covered by warranty for a period of 3 years. Damage through incorrect use will invalidate this warranty.

Cushion Cover

All Harvest Healthcare Ltd cushion covers are covered by warranty for a period of 12 months. Damage through incorrect use and penetration by sharp instruments will invalidate this warranty.

GUARANTEE

Harvest Healthcare Ltd guarantees to repair or replace all goods issued to its customers which are found to be defective whilst still under their applicable warranty period. All warranties are subject to our standard terms and conditions of sale which can be found on our website or by request to Harvest Healthcare Ltd.

If this device malfunctions and a serious incident occurs please contact Harvest Healthcare Ltd or your competent authority. A member of our service team will be happy to help you and offer advice.

Email:sales@harvesthealthcare.co.ukTelephone:01709 377172

DECLARARTION OF CONFORMITY

DECLARATION OF CONFORMITY

Declaration of Conformity Annex VII EU Directive 93/42/EEC

We, as company: Harvest Healthcare Ltd Sheaf House Bradmarsh Way Bradmarsh Business Park Rotherham, S60 IBW

Authorised Representative: TBC SRN: TBC

Confirm on our own behalf that the medical product:

Please see the Static Cushions list on the next page. Risk Class: I SRN: TBC

complies with all applicable requirements in Appendix I of the EU directive 93/42E EC The following compliance evaluation process was applied: Annex VII EU legislation for this type of product covers all safety, health and environmental requirements.

Modifying this product without consultation with Harvest Healthcare Ltd will invalidate this declaration of conformity.

This declaration of conformity is issued under the sole responsibility of Harvest Healthcare.

Notified Body: TBC

Jahn u. Cm

Product Development and Quality Manager Rotherham, 08.04.2020

This document is valid for 4 years after the date of the signature.

NOTES

DECLARATION OF CONFORMITY

Declaration of Conformity Annex VII EU Directive 93/42/EEC

Here is the list of the Static Cushions.

HFC121	UDI: 5060517530303
HFC116	UDI: 5060517530310
HFC128	UDI: 5060517530327
HFC128.1	UDI: 5060517530334
HFC128.2	UDI: 5060517531546
HFC129.01	UDI: 5060517532147
HFC129	UDI: 5060517530341
HFC123	UDI: 5060517530358
HFC130	UDI: 5060517530402
HFC132	UDI: 5060517531065
	HFC116 HFC128 HFC128.1 HFC128.2 HFC129.01 HFC129 HFC123 HFC130



HH20 - ACTIVE SYSTEM

This one-piece alternating cushion has been developed to provide maximum patient comfort whilst achieving optimum pressure relief, whilst seated.

The Active Seat System provides relief/pressure reduction during periods out of bed. This inexpensive system provides relief and comfort for those at risk of developing a pressure ulcer as well as assisting in the healing of existing problem areas.

It is compatible with all of Harvest Healthcare's pumps.

Please note that this is a cushion only, pumps are available separately.

For more information, please contact a member of the sales team on 01709 377 172

	MED	HIGH	V HIGH		
PRESSURE RELIEF RISK					

Serial No:

DOCUMENT REFERENCE: CUSHIONS/MANUAL - Version 2 - April 2020



Harvest Healthcare Limited. Company No: 07210261 Sheaf House, Bradmarsh Way, Bradmarsh Business Park, Rotherham, S60 1BW

T +44 (0)1709 377 172 F +44 (0)1709 377 173 E sales@harvesthealthcare.co.uk www.harvesthealthcare.co.uk

© Copyright Harvest Healthcare Our Full Terms & Conditions are available by request or can be found on our Website. Harvest Healthcare reserves the right to alter or amend this document without notice.