

# ANTI-ENTRAPMENT BUMPER & EXTENSION PIECE



## Product Code:

HCSAEBUMP791	Anti-Entrapment Bumper with Net/Mesh Inserts for Woburn & Community Bed (L: 2000 mm)
HCSAEBUMP140	Anti-entrapment Bumper with Net/Mesh Inserts for Beds with 3 Bar Side Rails (Ultimate/Safeguard) (L: 2000 mm)
HCSAEBUMP791 + HCSA -BUMP791.EX20	Extended Anti-Entrapment Bumper with Net/Mesh Inserts for Woburn & Community Bed (L: 2200 mm)
HCSAEBUMP140 + HCSA -BUMP140.EX20	Extended Anti-Entrapment Bumper with Net/Mesh Inserts for Woburn Ultimate Bed (L: 2200 mm)

## Description:

- The Anti-Entrapment Bumper is a 2 bar bumper with net/mesh insert and anti-entrapment system with link sheet.
- Designed for users who move excessively in bed, to prevent limbs from becoming trapped.
- The Anti-Entrapment Bumper fits all standard width Woburn beds, whilst also being suitable for most other manufacturer's beds with two full length wooden side rails at each side of the bed.
- We also offer the Anti-Entrapment Bumper for beds with 3 bar side rails (Ultimate/Safeguard).
- All the bumpers come with covers that are removable and machine washable, for better infection control.

## Cleaning and Care:

The cover is removable and machine washable, for better infection control.  
 Pre wash cold 10 minutes  
 Main wash 80°C 10 minutes  
 Followed by cold rinses and extraction.  
 Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.  
 Tumble dry at low heat.  
 Do not iron.  
 Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

## Latex free:

Yes

## User Instruction:

Yes

## Flammability:

BS 7175: 1989 (2013) ignition sources 0,1



**2 Bar Anti Entrapment Bumper**



**3 Bar Anti Entrapment Bumper**



**Extended Anti-entrapment Bumper  
with Net/Mesh Inserts**

**Supplier Contact Details:**

t: +44(0)1709 377172

e: [enquiries@harvesthealthcare.co.uk](mailto:enquiries@harvesthealthcare.co.uk)

w: [www.harvesthealthcare.co.uk](http://www.harvesthealthcare.co.uk)

Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

**Important Safety Notice**

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products.

Products and data continually undergo development and may therefore deviate from the details given.

**Version 1: February 2022**