

BED SIDE PLATFORM BUMPER



Product Code: HWB004

Description:

- The Bed Side Platform Bumper has been designed to easily fit onto the outer edge of the bed frame.
- To reduce the risk of injury for patients on the frame when getting in and out of bed.
- Suitable for use on all our Woburn bed range and similar types of profiling beds produced by other manufacturers.
- Created using soft to the touch material, protecting the user and increasing patient safety.

Cleaning and Care:

The cover is removable and machine washable, for better infection control. Pre wash cold 10 minutes
Main wash 80°C 10 minutes
Followed by cold rinses and extraction.
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
Tumble dry at low heat.
Do not iron.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

User Instruction: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Harvest Healthcare Ltd supplies a range of products such as foam mattresses, active air mattresses, fall protection mats and bed rail protection bumpers. These products are usually covered in vapour permeable PU fabric that could cause risk of suffocation in the event of prolonged facial contact.

(Mattress risk categories are to be used as a guide only)

Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

Version1: Feb 2022