

UNIVERSAL SAFE ROLL BUMPER



Product Code: HFB003

Description: Universal Safe Roll Bumper

- The Universal Safe Roll system is a state-of-the-art crash mat, designed to dramatically change the landscape of falls protection and support the reduction of the use of side rails. Traditional static floor mats are often a trip hazard, have to be moved to accommodate moving and handling equipment, and are frequently too thin to provide true protection from falls. This radical new design eliminates all these issues, providing safety for the resident and exceptional ease of use for carers.
- The folding mat attaches to the side of the bed, and Hook and Loop up when the bed is raised for care (image 1), enabling access for carers and for equipment.
- As the bed is lowered the deep foam mat folds out to lie alongside the bed, as a true safety feature (image 2).
- Designed to fit our Woburn Ultra Low, Low and standard beds, the Universal Safe Roll is suitable for a wide range of users and is ideal for use within nursing homes and the domestic environment.
- All the bumpers come with covers that are removable and machine washable, for better infection control.



Image 1



Image 2

Cleaning and Care: The cover is removable and machine washable, for better infection control. Pre wash cold 10 minutes
Main wash 80°C 10 minutes
Followed by cold rinses and extraction.
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
Tumble dry at low heat.
Do not iron.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

User Instruction: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk
Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products.
Products and data continually undergo development and may therefore deviate from the details given.

Version 1: February 2022