

UNIVERSAL DELUXE SLING





Product Code:

HSL100

HSL100.SS (Super Soft) HSL110 (With Clips)

HSL300 (With Head Support)

HSL300.SS (Super Soft With Head Support)

Description:

- The Universal Deluxe Sling is a hybrid sling designed to offer the full support of a hammock sling with the function of a universal
- Can be applied and removed while seated in a chair
- Full padded support of the upper thigh gives maximum comfort, lifting from both inner and outer thigh to reduce hip rotation and abduction
- Suitable for the majority of users including amputees and those prone to excessive movements
- Woven label to increase the life of the sling
- Pocket for ease of fitting
- UK manufactured
- Extra stitching to increase the quality and safe working load
- Available in a range of sizes
- Available with padded head support
- Secured by loop or clip fastening
- Available in Polyester and Super Soft

Safe Working Load:

400kg / 62 Stone (Polyester) 300kg / 47 Stone (Super Soft)

Cleaning Instructions: Each sling can be laundered up to 80°c using a mild soap solution and then rinse. Slings will air dry quickly in a warm room or in the normal manner. Tumble dry on low setting only. Do not place over a convector heater or steam pipes. Do not use Biological detergents. Do not use Bleach. Do not use strong chemicals. A faded label is an early indicator that sling is subject to harsh / abrasive cleaning methods do not continue to use sling. All slings fitted with a Hook and Loop body belt should have the Hook and Loop square removed during laundering and refitted afterwards. This will significantly extend the lifespan of the product.

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Harvest Healthcare Ltd supplies a range of products such as foam mattresses, active air mattresses, fall protection mats and bed rail protection bumpers. These products are usually covered in vapour permeable PU fabric that could cause risk of suffocation in the event of prolonged facial contact. (Mattress risk categories are to be used as a guide only)

sments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given. Version