

# SAFETY WEDGE



**Product Code:**

**HCS096:** Safety Wedges with Double Duty Hook and Loop Base Sheet

**HCS096B:** Safety Wedges 900x900mm with Hook and Loop Loop

**Description:**

- The Safety Bed Wedges provide a safe, firm, sloping barrier to keep the user comfortably positioned whilst in bed.
- The wedges are connected to a waterproof base sheet meaning the product is easy to fit and will be secure once fitted onto the bed.

**Cleaning and Care:**

The cover is removable and machine washable, for better infection control. Pre wash cold 10 minutes  
Main wash 80°C 10 minutes  
Followed by cold rinses and extraction.  
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.  
Tumble dry at low heat.  
Do not iron.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

**Latex free:**

Yes

**User Instruction:**

Yes

**Flammability:**

BS 7175: 1989 (2013) ignition sources 0,1

**Important Safety Notice**

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Harvest Healthcare Ltd supplies a range of products such as foam mattresses, active air mattresses, fall protection mats and bed rail protection bumpers. These products are usually covered in vapour permeable PU fabric that could cause risk of suffocation in the event of prolonged facial contact.  
(Mattress risk categories are to be used as a guide only)

Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

**Version1: Feb 2022**