

STANDARD BUMPER & EXTENSION PIECE



Product Code:

HCSBUMP112	Standard 2 Bar Bumper with Net/Mesh Inserts for Woburn, Community & 1200 Bed (L: 2000 mm)
HCSBUMP212	Extended 2 Bar Bumper with net/mesh inserts for Woburn, Community & 1200 Bed (L: 2200 mm)
HCSBUMP130	Standard 3 Bar Bumper with Net/Mesh Inserts for Woburn Ultimate Bed (L: 2000 mm)
HCSBUMP112 + HCSBUMP112.EX20	Extended Standard 2 Bar Bumper with Net/Mesh Inserts for Woburn, Community & 1200 Bed (L: 2200 mm)
HCSBUMP130 + HCSBUMP130.EX20	Extended Standard 3 Bar Bumper with Net/Mesh Inserts for Woburn Ultimate Bed (L: 2200 mm)

Description:

- The Standard Bumper with net/mesh inserts is designed to fit over side rails, providing a safe protection for users in bed.
- The net/mesh inserts are entwined with the bumpers, to improve visibility between each side rail, reducing the feeling of entrapment.
- The bumper fits all Woburn beds.
- All the bumpers come with covers that are removable and machine washable, for better infection control.

Cleaning and Care:

The cover is removable and machine washable, for better infection control.
 Pre wash cold 10 minutes
 Main wash 80°C 10 minutes
 Followed by cold rinses and extraction.
 Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
 Tumble dry at low heat.
 Do not iron.
 Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free:

Yes

User Instruction:

Yes

Flammability:

BS 7175: 1989 (2013) ignition sources 0,1



2 Bar Anti Standard Bumper



3 Bar Standard Bumper



Extended Standard Bumper with Net/Mesh Inserts

Supplier Contact Details:

t: +44(0)1709 377172

e: enquiries@harvesthealthcare.co.uk

w: www.harvesthealthcare.co.uk

Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products.

Products and data continually undergo development and may therefore deviate from the details given.

Version 1: February 2022