

WRAPOVER BUMPER



Product Code:	HCSWRAP113	Harvest Wrapover Bumper Standard
	HCSWRAP113 + HCSWRAP113.EX20HEIGHT	Harvest Wrapover Bumper Standard & Height Extension (200mm)
	HCSWRAP113 + HCSWRAP113.EX50	Harvest Wrapover Bumper Standard & Length Extension (500mm)

- Description:**
- The Wrapover Bumper is used to surround the bed side rails, providing a safe and cushioned barrier for patients.
 - Designed with users in mind who are inclined to roll whilst in bed, meaning extra protection may be necessary.
 - The three quarter length bumper wraps over the side rails and secures with Hook and Loop at the bottom, ensuring ease of use.
 - All the bumpers come with covers that are removable and machine washable, for better infection control.
 - The 200mm height extension suitable for the Wrapover Bumper is used to surround the bed side rails, providing a safe and cushioned barrier for patients. Designed for use when the height attachment side rails are placed onto the bed, ensuring full protection for the user.
 - The 500mm length extension suitable for the Wrapover Bumper is used to surround the bed side rails, providing a safe and cushioned barrier for patients. Designed for use when the bed has been extended, ensuring full protection for the user.

Cleaning and Care: The cover is removable and machine washable, for better infection control.
 Pre wash cold 10 minutes
 Main wash 80°C 10 minutes
 Followed by cold rinses and extraction.
 Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
 Tumble dry at low heat.
 Do not iron.
 Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

User Instruction: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1



Harvest Wrapover Bumper Standard & Length Extension (500mm)

Supplier Contact Details:

t: +44(0)1709 377172

e: enquiries@harvesthealthcare.co.uk

w: www.harvesthealthcare.co.uk

Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

Version 1: February 2022