



harvesthealthcare®
a prismhealthcare company

CONTACT PAD



Product Code: HCS140CONTP

- Description:**
- Designed for use with the Harvest Anti-Entrapment Bumper, the contact pads protect users with involuntary movement from skin damage and prevents the risk of friction injury from contact with the mesh inserts.
 - The pads can be positioned anywhere inside the bed with a 'hook and loop' fastening.
 - All the bumpers come with covers that are removable and machine washable, for better infection control.

Cleaning and Care: The cover is removable and machine washable, for better infection control. Pre wash cold 10 minutes
Main wash 80°C 10 minutes
Followed by cold rinses and extraction.
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
Tumble dry at low heat.
Do not iron.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk
Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

Version 1: March 2022