



General User/Safety Guide

SNUG WALL BUMPER- HWB003

SNUG WALL
BUMPER



www.harvesthealthcare.co.uk

CE
UK
CA

CONTENTS

WARNINGS & CAUTIONS	4
PRODUCT OVERVIEW	5
GENERAL INFORMATION	6-7
INSTALLATION	8-10
CLEANING AND CARE	11

WARNINGS & CAUTIONS



READ THIS INSTRUCTION MANUAL CAREFULLY. PLEASE NOTE IN PARTICULAR THAT THE SAFETY INSTRUCTIONS MUST BE OBSERVED.



WARNING

Please ensure that this booklet is read prior to first use.

Please refer to the information in this booklet as a guideline only.

Harvest Healthcare Ltd recommend that prior to using this product, a full risk assessment must be completed by a qualified professional, in order to determine that the correct product is suitable for each individual.

Do not allow sharp objects to penetrate the Snug bumper or the covering material.

Ensure this product is fitted as specified in the fitting instructions.

Routinely inspected this item for damage and ensure all fixing straps or Hook and Loop are securely fastened.

Remove the item from service if the product is damaged or ripped.

Remove the item if the fixing straps or the Hook and Loop show signs of wear and they are not effective.

Do not expose this product to open flames. We advise the user not smoke when using this product.

Facial contact with the cover may cause suffocation.

SYMBOLS USED



COVERS ARE
REMOVABLE AND
MACHINE WASHABLE



LOW HEAT
TUMBLE DRY



DO NOT
DRY CLEAN



DO NOT
BLEACH



DO NOT IRON



CE MARK



UKCA MARK



DO NOT USE
PHENOLS



NO
SMOKING



This symbol indicates general hazards. There is a danger to life and health.

PRODUCT OVERVIEW

PRODUCT CODE: HWB003

PRODUCT DESCRIPTION

Snug Wall Bumper for profiling beds

The Snug wall bumper has been designed to fit onto our Woburn bed range or other similar types of profiling beds produced by various manufactures.

Beds often need to be placed slightly away from the wall due to room fittings and fixtures, which can leave a gap. This can be an entrapment risk, and the Snug bumper unobtrusively and comfortably fills this space. This removes the entrapment risk without the need for side rails to be fitted.

The bumper helps to prevent accidental contact with the wall, the material we use is soft to the touch giving the end user reassurance and protection.

Ideal for nursing homes and care in a domestic environment. The wall snug is easy to fit, simply place and fasten the bumper to the side of the bed which will be against the wall. Push the bed up against the wall leaving a slight gap so the bed can be raised and lowered safely. Refer to the fitting instructions.

After fitting the snug wall bumper check that all the brakes on the bed are applied to prevent any movement of the bed when it is being operated.



GENERAL INFORMATION

INSTRUCTIONS FOR USE

Check the product for damage, and it is the correct type product for the bed, before carrying out the installation.

- If a Snug bumper or any other part of the product looks damaged do not use the bumper.
- Refer to the product code and pictures to make sure you're using the right instruction for use.
- Before installing this product read the instruction for use thoroughly and seek advice if you are unsure how to fit the product.

To help prevent injury it is important that patients and equipment are monitored regularly.

The Snug bumper may be delivered in a plastic cover for protection; these covers must be removed and disposed of before the equipment can be used.

Please dispose of the plastic as specified by your local waste disposal service authority.

INTENDED PURPOSE

The bumper helps to prevent accidental contact with the wall, this is an alternative to using a side rail system which might not be suitable for the bed or it could cause injury to the end user if they are prone to involuntary movements.

SNUG WALL BUMPER SYSTEM INSPECTION RECOMMENDATIONS

The Snug Wall system should be checked regularly to ensure they remain 'fit for purpose', effective and pose no risk of infection to either the patient or the carer.

Harvest Healthcare Ltd recommend that a thorough inspection of both the exterior (cover) zips, straps and Hook and Loop is carried out weekly, or each time a new patient is introduced to this equipment. Visual checks should be carried out daily to identify any significant signs of damage or infection risk.

If any signs of contamination are identified, the side rail system should be withdrawn from use immediately and cleaned.

CLEANING AND CARE

The side rail system has a mesh and protective PU cover. The cover should be cleaned regularly in accordance with your organisation's disinfection protocol.



Facial contact with any covering material such as that used on this product can cause suffocation and a Risk Assessment should be done prior to the use of this product.

GENERAL INFORMATION

MOVING, HANDLING AND STORAGE

Where possible, the bumper should be stored flat to prevent damage to the product.

It may require more than one person to aid in fitting the system.

The bumper is to be stored between -5 to +40 degrees, in a dry environment.

Do not expose to high humidity or heat.

Do not drag or pull the bumper system by the cover.

SAFE USE

This product is intended as an aid to prevent accidental injury. With a carer present check the material, zips and fastenings to ensure that these parts of the bumper systems have not been damaged and do not introduce gaps too or a hazard that could be increased the risk of entrapment suffocation or injury to the end user or carer.

Before and during the use of the bumper, consider the effectiveness of the system. The hazard of entrapment can still occur if the bumper has not been fitted correctly or not attached to the bed.

Seek advice to ensure the product is suitable for the bed before attempting to fit it. The fastening straps must loop around the static parts of the bed. Thoroughly read the fitting instructions supplied with this product.

Important Notice

Do not sit on the bumper as this will deform the product and could cause result in less protection being provided to the end user.

The bed braking system must be applied when using this product.

WARRANTY


Bumper Systems

All Harvest Healthcare bumpers are covered by warranty for a period of 1 year. Damage through incorrect use will invalidate this warranty.

GUARANTEE

Harvest Healthcare Ltd guarantees to repair or replace all goods issued to its customers which are found to be defective whilst still under their applicable warranty period. All warranties are subject to our standard terms and conditions of sale which can be found on our website or by request to Harvest Healthcare Ltd.

INSTALLATION

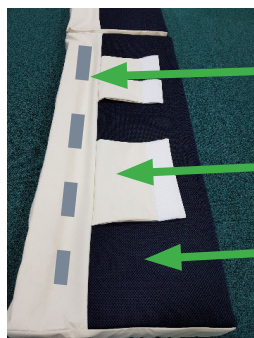


Recess to allow installation to the sleeping platform.

Hook and Loop flaps to secure the system to the sleeping platform.


Spacer mesh material to reduce the risk of suffocation.

1. Open all the Hook and Loop flaps.

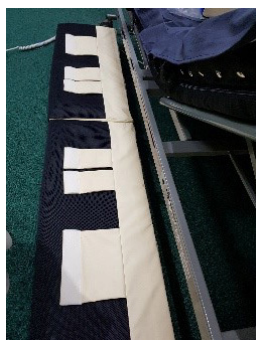

2. Present the bumper system to side of the bed and locate the edge of the sleeping platform in the recess in the wall bumper.


3. Fold the Hook and Loop flaps over the edge of the platform and secure on the underside of the bumper.

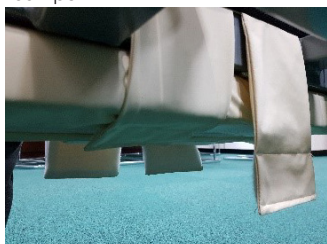

The system is purposefully designed to be non-self-supporting. Without a wall at the side of the bed the bumper will roll outwards from the bed.

INSTALLATION

4. Lift up the bumper and move the bed towards the wall, it will fill the gap between the wall and mattress where side rails would normally fit.

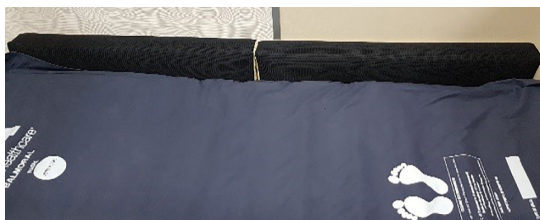

Please note

- Carry out a risk assessment to ensure it is suitable for the users' requirements and is compatible with the bed.
- Please take care to avoid any obstacles on the wall that could damage the bumper system or injure the bed user.
- These bumpers use the wall as support, the bed must be against a wall when left unattended.
- The outer cover can be removed for decontamination.
- In the event of the bumper system becoming damaged discontinue use immediately and contact Harvest Healthcare for advice.

CLEANING & CARE



WARNING

Eye protection, gloves and protective clothing should be worn when carrying out cleaning and disinfection procedures.

When disinfecting the system, Harvest Healthcare recommends the following guidelines which have been developed to comply with recognised infection control procedures. These procedures are also to be used to prevent cross infection when transferring the system between patients.

SNUG WALL BUMPER SYSTEM MATERIAL COVER

During general use the cover can be cleaned by wiping it with a mild detergent solution. The netting wipe with soap and water or a with a sodium hypochlorite solution diluted to 1000ppm and tumble dry at low heat or allow to naturally dry.



Do not use abrasive cleaners, phenol disinfectants, solvents or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.



Do not iron.



Ensure that the side rail netting material is thoroughly dried before refitting the bumper or placing in storage.



HYPERCARBONATE AND PHENOL-BASED SOLUTIONS SHOULD NOT BE USED AS THESE WILL DAMAGE THE MATTRESS COVER

DISPOSAL

If contaminated, please dispose of as clinical waste.

If not contaminated dispose of as household waste at your local approved facility.

Note:

Please report any serious incident related to this device to Harvest Healthcare.

Full contact details can be found at the end of this document.

NOTES



Sheaf House, Bradmarsh Business Park,
Bradmarsh Way, Rotherham S60 1BW

T +44 (0)1709 377172

F +44 (0)1709 377173

E sales@harvesthealthcare.co.uk

www.harvesthealthcare.co.uk

© Copyright Harvest Healthcare - HWB003 Snug Wall Bumper Manual - Version 2
July 2022

Harvest Healthcare Limited. Company No: 07210261

Our Full Terms & Conditions are available by request or can be found on our Website.
Harvest Healthcare reserves the right to alter or amend this document without notice.