

SNUG WALL BUMPER



Product Code: HWB003

Description: Snug Wall Bumper

- The snug wall bumper has been designed to fit onto our Woburn bed range or other similar types of profiling beds produced by various manufacturers.
- The product fills the need to provide a safe environment for the user who may not need side rails fitting.
- The bumper helps to prevent accidental contact with the wall, the material we use is soft to the touch giving the end user reassurance and protection.
- Ideal for nursing homes and care in a domestic environment.
- It is easy to fit. Fitting instruction video is available on the website.

Note: After fitting the snug wall bumper check that all the brakes on the bed are applied to prevent any movement of the bed when it is being operated.

Cleaning and Care: The cover is removable and machine washable, for better infection control.
Pre wash cold 10 minutes
Main wash 80°C 10 minutes
Followed by cold rinses and extraction.
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
Do not iron.
Tumble dry at low heat.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

User Instruction: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk w: www.harvesthealthcare.co.uk
Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Risk assessments specific to user must be carried out prior to the use of all products. Products and data continually undergo development and may therefore deviate from the details given.

Version 2: March 2