




New Bed Accessories

Convenient accessories you didn't know you needed!

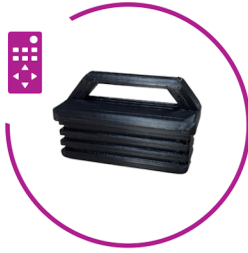

HANDSET HOLDER

Always have your bed's control handset safely within reach. Effortlessly attaches to your Woburn Community bed to conveniently hold your handset.

Product Code: [HLDW008](#)



REUSABLE CABLE RETAINER

A simple cable management solution. Keep your cables neatly and safely out of the way.

Product Code: [HLB603](#)



PUMP GRIPS

Keep your pump secured across a range of different headboards. Available in 2 sizes:

- Small (Harvest Pump 2): [PU215](#)
- Large (Harvest Pump 1): [PU115](#)

NEW!

DESIGNED AND DELIVERED IN-HOUSE

These new products have been designed and manufactured in-house by Harvest Healthcare. We use our on-site technology to develop exciting, innovative bespoke parts and accessories. We are continually expanding our product range to deliver convenient solutions for the modern care environment.

Please get in touch for any bespoke manufactured solutions. Please see overleaf for contact information.



Bed Accessories Guide

How to use your new accessories.



HANDSET HOLDER

Simply remove the plastic cover from the frame of the bed and replace it with your handset holder by pushing firmly until secure. The Holder can be placed in any four corners of the bed frame allowing it to be wherever is most convenient for you. Compatible with the full Community range.

 <https://youtu.be/vI7jZ6ciZwM>



REUSABLE CABLE RETAINER

To set it up all you have to do is twist the end of the retainer until unlocked then place the loose wires and cables inside then simply twist it back into place. Attach the cable tie onto a static section of the frame on the bottom of the bed and then you are done. Excellent for keeping areas under the bed tidy and secure.

 https://youtu.be/XCYR_iqsGbw



PUMP GRIPS

Simply place your Harvest pump on your bed footboard and push the hangers closer together until the pump is secured to the bed. Then place the ends of the hangers into either side of the holes in the grips and adjust accordingly until secure. This will ensure that your pump is safely secured on your bed.

 <https://youtu.be/bOX24bDgnWg>



GET IN TOUCH

To order, or for more information on our bed accessories range, please get in touch with our friendly and informative team or alternatively visit our website.



+44 (0)1709 377172



www.harvesthealthcare.co.uk



sales@harvesthealthcare.co.uk