


Before you can use the Guardian falls alarm system you must complete the full set up including pairing and sequencing. The quick reference guide assumes the system has already been set up as described in the full manual.


POWER

Install 3 AAA batteries (not supplied) and/or connect the power supply.



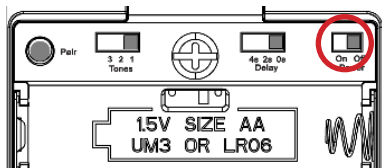

- Turn on the receiver using the switch inside the battery compartment at the right-hand side top corner.

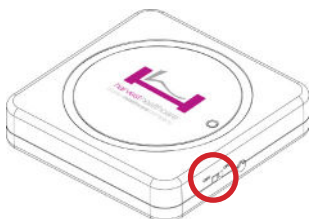



- Turn on the transmitters (using the small switch at the side of the transmitter).



- Allow the initialisation sequence to run. (See display examples page 2.)
- Once initialised, check each of the transmitters is paired and has good signal/battery. (See display status examples 8 and 9.)

| Diagram | Description |
|---|--|
|  | Paired if flashing indicates paired but no connection to transmitter) (Transmitter Low Battery) |
|  | Alarm In Use (Transmitter Low Battery) |
|  | Pad In Use (Transmitter Low Battery) |
|  | Signal Loss |

- Install the required sensor mats (see the full instruction manual).
- Each time the floor mat is triggered the receiver should alarm immediately and trigger the nurse call system (if connected). (See display status examples 5 and 6.)
- The seat and mattress sensors will make the receiver sound a single tone when a person is placed on the sensor mat. When the person moves from the sensor mat the receiver will alarm and trigger the nurse call system (if connected).
- Cancelling an alarm
 - Short press  to cancel the current alarm (cancel one at a time if there are multiple alarms).
 - Long press for  3 seconds to cancel all alarms.

Important: Cancel the alarm on the receiver before cancelling the alarm on the nurse call (if connected).


For the full installation guide and detailed information see the full IFU.

GUARDIAN WIRELESS FALLS ALARM SYSTEM QUICK REFERENCE


Display Status Examples




1. Receiver Battery Low
Replace battery now or use mains power supply

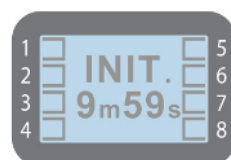

2. System Initialising
System may not function correctly while this is displayed

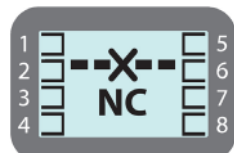

3. Nurse Call Cable Disconnect
Check cable from receiver to nurse call

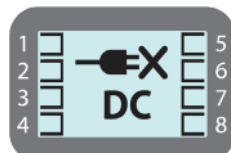


4. DC (Mains Voltage) Power Fail
Check power supply




5. Trigger From Transmitter 1
Mattress or Seat Sensor Mat




6. Trigger From Transmitter 1
Floor Sensor Mat

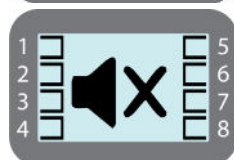

7. Receiver Set to Silent Alarm
Long press button

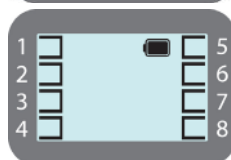

8.
Receiver battery full
Receiver battery
Receiver battery low

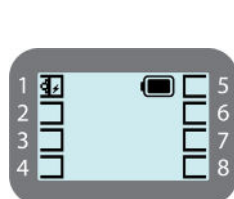

9.

- Paired
(Transmitter Low Battery)
- Alarm In Use
(Transmitter Low Battery)
- Pad In Use
(Transmitter Low Battery)

Testing the System

Floor Sensor Mat

Apply pressure to the sensor mat, the receiver should trigger a full alarm immediately and indicate the transmitter information on the receiver (see display status examples 6)

If the system does not trigger wait 5 seconds and then remove pressure from the floor sensor mat.

If the mat triggers when you remove pressure check you are using the correct sensor mat.

Seat or Chair Sensor Mat

Apply pressure to the sensor mat, the receiver should sound a double tone.

Remove pressure from the sensor mat and the receiver should trigger a full alarm as soon as pressure is removed and indicate the transmitter information on the receiver (see display status examples 5)

Testing the Connection to the Nurse Call System

To test the receiver is correctly connected and configured to the nurse call system hold and together to trigger an alarm.

The nurse call system should trigger as soon as the receiver alarms.

If the nurse call system triggers when the alarm on the receiver is cancelled change the position of the NC/NO selector (see full user manual or contact Harvest Healthcare).

The most common setting for this switch is N/O.

Please Note

- Carry out a risk assessment to ensure the Guardian System is suitable for the intended application.
- To be installed and operated by competent persons who have read and fully understood the full IFU (supplied with each system).
- Test the system for correct operation daily and before each use.
- Pay attention to the display on the receiver for any warning notifications.
- If you have any doubts about installation or operation please call Harvest Healthcare before using the system.