

Quantec Nurse Call System Connections

HJR075 Alarm System



HLV006 Y Connector

HJM330.S
Nurse Call Handset

HJM310 cable

Connect the stereo
end towards the
nurse call system

HJM312 Cable

HLV006 Y Connector

HLV301M

Mattress Pad Sensor


HLV301S

Seat Pad Sensor



The HJM301 FLOOR pressure pad can be plugged directly into the nurse call system (does not require the HJR075 alarm system) by using the HJM312 cable

HJM301 Floor Pad Sensor

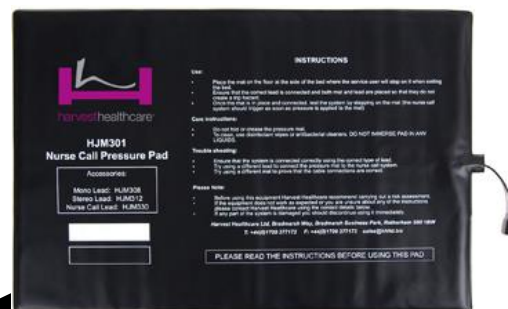

To Use a Seat and or Mattress Sensor Pad

1. Connect a HLV006 Y connector into nurse call system
2. Connect nurse call handset to one connector and a HJM309 cable to the other connector
3. Connect the other end of the HJM309 cable into the HJR075 alarm system socket marked NURSE CALL
4. Connect another HLV006 Y connector to the black cable already connected to the Alarm system.
5. Connect the HLV301M mattress pad to one connector and the HLV301S seat pad to the other.
6. If the chair is located away from the bed you can use a HLV006E 6-meter extension cable

Quantec Interface Unit

If you have the interface socket (pictured below right) fitted alongside the standard nurse call (pictured below left) You can plug a seat pad sensor **or** floor pad sensor directly into the interface. Ensure you have correctly identified the interface and test before using on a resident.



Standard Nurse Call Socket



Optional Interface

- Carry out a risk assessment to ensure this system is suitable for intended use and test system for correct operation each time before use.
- If using a seat pad and mattress pad only one can be in use at any time.
- Read the HJR075 alarm system instructions carefully.
- If you have any doubts about the above please contact Harvest Healthcare Ltd.

Stereo Jack-Plug

A stereo jack-plug can be identified by the 2 black insulation bands.



Mono Jack-Plug

A mono jack-plug can be identified by the single black insulation band.

HJM312 CABLE

6.35MM stereo jack-plug with 3.5mm mono jack-plug.

Used to connect the floor mat to the nurse call system.



HJM310 CABLE

6.35MM stereo jack-plug with 6.35mm mono jack-plug.

Used to connect the HJR075 alarm system to the nurse call system.



HJM330.S

6.35MM stereo jack-plug with to nurse call button



HLV006 Y CONNECTOR

6.35mm stereo jack-plug to 2 x 6.35mm sockets.

Used to connect either mat and nurse call button to nurse call system.

Or seat and mattress sensors to nurse call system.

