



General User/Safety Guide

PIVOTPRO OVER BED TABLE

BED ACCESSORIES



www.harvesthealthcare.co.uk

CE
UK
CA

CONTENTS

WARNINGS & CAUTIONS	3
GENERAL INFORMATION	4
OVERVIEW - PIVOTPRO OVERBED TABLE	5
FITTING THE PIVOTPRO OVERBED TABLE TO A WOBURN BED	6-7
FITTING THE PIVOTPRO OVERBED TABLE TO OTHER BEDS	8-10
USEFUL PART LIST	11

WARNINGS & CAUTIONS



READ THIS INSTRUCTION MANUAL CAREFULLY.



PLEASE NOTE IN PARTICULAR THAT ALL THE SAFETY INSTRUCTIONS MUST BE OBSERVED.



WARNING

The PivotPro overbed table has a maximum Safe Working Load (SWL) of 8kg. **Never** sit or lean on the table and never use it as a positioning aid.

The PivotPro overbed table should be positioned at the center of the sleeping platform. There should always be a distance of less than 60mm from the headboard or more than 318mm from the headboard. Failure to fit the PivotPro overbed table in the correct position may cause a risk of entrapment.

The PivotPro overbed table should never be used with side rails.

The PivotPro overbed table and its fixings must be checked for any defects after each use and once again before each new installation, once installed the fixings should be checked for security at regular intervals.

Before fitting the overbed table, it is important that an individual risk assessment is performed for each user on a case-by-case basis. This is required to assess both the suitability of the PivotPro for a particular client and the compatibility between the PivotPro and the bed it is being fitted to as well as other accessories/equipment being used.

The PivotPro overbed table has a SWL of 8kg and is designed to hold objects up to this weight. It is not designed to be sat/climbed or leaned on or pulled when repositioning the bed. Never exceed the SWL.

Beware of placing hot items on the table as they could cause injury to the person in the bed if they are knocked over.



CAUTION

Ensure the PivotPro overbed table is suitable for the bed it is being fitted to and installed in the correct position.

Ensure the securing fixings are tight and are checked at regular intervals. Do not allow the PivotPro to cause an increased risk to the patient.

Care must be taken with raising the backrests, as the gap between the two components could become less than 25mm in certain situations.

It is recommended to remove the mattress guide closest to the PivotPro overbed table to prevent any collision between the PivotPro overbed table and the mattress guides.

The PivotPro overbed table and all components should be stored in a warm dry environment when not in use.

When operating the profile bed sections, pay attention to the position of the PivotPro overbed table to ensure the bed user is not caught between the moving bed sections and the overbed table.

If you have any doubt about the suitability or fitting of the PivotPro overbed table, contact Harvest Healthcare Ltd. Report any issues involving the use of this overbed table to Harvest Healthcare Ltd.

GENERAL INFORMATION

EXPLANATION OF THE SYMBOLS USED IN THE IFU AND THE LABELS.



Read instructions / consult manufacturers guide.



This symbol indicates general hazards. There is danger to life and health.



Conformity mark in accordance with the Medical Device Directive (93/42 EEC).



Declaration of Conformity for the UK



Fragile, handle with care.



This way up.



Keep dry.



Recycling symbol. Refers to packaging that can be recycled (cardboard).



Operation and storage temperature.



UK Representative.



Distributor



Maximum permissible load.



Manufacturer of the medical device



Manufacturing date



Medical Device

OVERVIEW


PIVOTPRO OVERBED TABLE



ON DELIVERY AND BEFORE INSTALLATION CHECK THAT THE PACKAGING IS UNDAMAGED. REPORT ANY VISIBLE DAMAGE TO THE TRANSPORT COMPANY IMMEDIATELY.

HLDWUL120- PIVOTPRO OVERBED TABLE

Weight	6.0kg
Clearance required under bed	Approx 10mm (4")
Safe Working Load	8kg

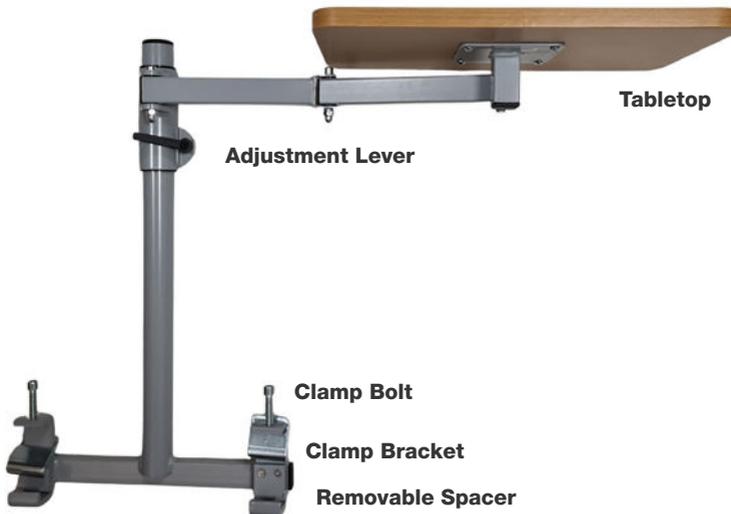

INSTALLATION PIVOTPRO OVERBED TABLE

FITTING THE PIVOTPRO OVERBED TABLE TO THE WOBURN BEDS

The following instructions show how to fit the PivotPro overbed table to a Woburn Bed.

- 1 Remove side rails (if fitted).
- 2 Ensure that the clamping bolts are loosened off and the clamping brackets are at the upper position.
- 3 Ensure the spacers are in place.
- 4 Harvest Healthcare recommend the PivotPro table is installed in the centre of the bed (avoiding any platform fixings). If it is installed in any position other than the centre of the bed it must be either less than 60mm or more than 318mm from the headboard. Failure to correctly position the PivotPro overbed table may pose a risk of entrapment.




Please pay attention to the sleeping platform fixings as the PivotPro overbed table will not fit over these fixings. Do not fit the PivotPro adjacent to any mattress guides. Remove the one closest to the PivotPro.




- 5 Tighten up the clamping bolt until the clamping brackets come into contact with the sleeping platform's bed outside frame. Ensure the clamping brackets are in contact with the sleeping platform and do not come into contact with any of the other fittings that could affect the fitting of the overbed table.

INSTALLATION PIVOTPRO OVERBED TABLE

- 6 Tighten the clamping bolts ensuring that the PivotPro overbed table is securely attached to the bed frame.

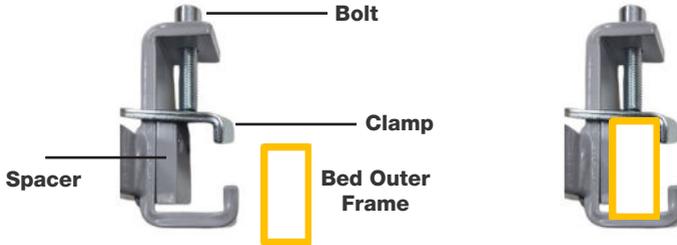

- 7 Adjust the position of the table (using the locking lever shown below) to ensure the table does not contact the bed user during operation, it's important the pivot is at the position shown below when used on the right side of the bed (it should be mirrored when used on the left side) This is to reduce the risk of the table being able to move over the backrest where it could collide if the bed is profiled and also give maximum coverage when the bed is used as an overbed table.
- 8 Raise the leg section on the bed fully and adjust the height of the table to ensure it will not hit the bed users legs.

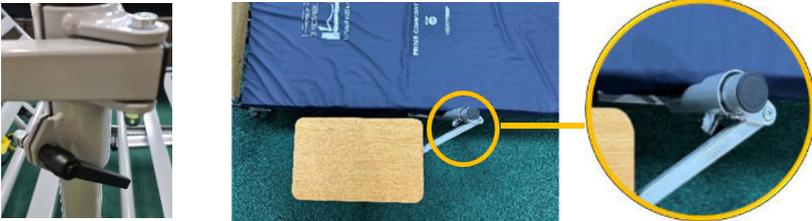

INSTALLATION PIVOTPRO OVERBED TABLE

FITTING THE PIVOTPRO OVERBED TABLE TO A BED OTHER THAN A HARVEST WOBURN BRAND.

- 1 Carry out a risk assessment to ensure the two products and any other equipment in use are compatible.
- 2 The spacers can be removed to allow the PivotPro overbed table to fit on beds where the platform frame tubes may be larger than the Harvest Healthcare Ltd range of beds.
- 3 Remove side rails (if installed).
- 4 Ensure that the clamping bolts are loosened off and the clamping brackets are at the upper position.
- 5 Harvest healthcare recommend the PivotPro table is installed in the centre of the bed (avoiding any platform fixings). If it is installed in any position other than the centre of the bed it must be either less than 60mm or more than 318mm from the headboard. Failure to correctly position the PivotPro overbed table may pose a risk of entrapment.




Please pay attention to the sleeping platform fixings as the PivotPro overbed table will not fit over these fixings. Do not fit the PivotPro adjacent to any mattress guides. Remove the one closest to the PivotPro.



INSTALLATION PIVOTPRO OVERBED TABLE

- Tighten up the clamping bolt until the clamping brackets meet the sleeping platform. Ensure the clamping brackets are in contact with the sleeping platform and not any of the mattress platform fixings.
- Tighten the clamping bolt ensuring that the PivotPro overbed table is securely attached to the bed frame.

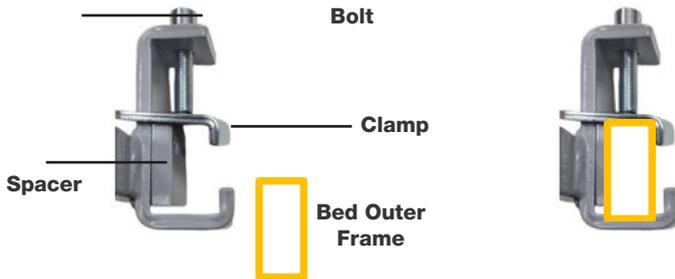

- Adjust the position of the table (using the locking lever shown below) to ensure the table does not contact the bed user during operation, it's important the pivot is at the position shown below when used on the right side of the bed (it should be mirrored when used on the left side) This is to reduce the risk of the table being able to move over the backrest where it could collide if the bed is profiled and also give maximum coverage when the bed is used as an overbed table.
- Raise the leg section on the bed fully and adjust the height of the table to ensure it will not hit the bed users legs.



- Pull the locking lever out and turn it to a position where it will not interfere with the person in the bed.
- Check the PivotPro is securely fixed to prevent unintentional movement, and profile the bed fully, to ensure there is no conflict with the moving platform sections and PivotPro overbed table.
- The PivotPro overbed table and its fixings must be checked for any defects after each use and once again before each new installation. Once installed the fixings should be checked for security at regular intervals.

INSTALLATION PIVOTPRO OVERBED TABLE

-  **13** The PivotPro overbed table has a SWL of 8kg and is designed to hold objects up to this weight and should never be exceeded or sat/climbed or leaned on.
-  **14** When operating the profile bed sections pay attention to the position of the overbed table to ensure the bed user is not caught between the bed sections and the overbed table and the patient is able to move and reposition safely.
-  **15** Be aware of placing hot items on the table, as they could cause injury if spilled.

The Pivot Pro table can be used in 2 positions.



Bedside Table



Overbed Table

USEFUL PART LIST

PIVOTPRO OVERBED TABLE HLDWUL120

SPARE PARTS

Clamping Bracket (Single)	HLB689
Clamp bolt	HLB690
Spacer Pad & Screws	HLB692



harvesthealthcare®
a prismhealthcare company



Sheaf House, Bradmarsh Business Park,
Bradmarsh Way, Rotherham S60 1BW
T +44 (0)1709 377172
F +44 (0)1709 377173
E sales@harvesthealthcare.co.uk
www.harvesthealthcare.co.uk



tecfor care GmbH
Fraunhofer Straße 8, 51647
Gummersbach Germany



Qarad UK Ltd.
8 Northumberland Ave, Westminster London
WC2N 5BY United Kingdom,
Company no. 13004437