

ULTRA LOW GRAB HANDLE



HLDWUL685 - ULTRA LOW GRAB HANDLE

Safe Working Load	60kg
Weight	4.0kg
Height from platform	610mm (24")
Width	240mm (9.5")



Fixing Bolt
(HLB690)

Spacer and 4 Screws (HLB692)

Clamping Bracket
(HLB689)



Includes x1
Alan Key
(HLB691)



Includes x2
Fixing Covers



READ THIS INSTRUCTION MANUAL CAREFULLY.



PLEASE NOTE IN PARTICULAR THAT THE SAFETY INSTRUCTIONS MUST BE OBSERVED.

CAUTION

The grab handle should either be positioned less than 60mm from the headboard or more than 318mm from the headboard. **Failure to fit the grab handle in the correct position may cause a risk of entrapment.**

If the grab handle is used on a bed with integral side rails beware of a potential pinch point between the grab handle and side rails.

The grab handle and its fixings must be checked for any defects after each use and once again before each new installation, once installed the fixings should be checked for security at regular intervals.

It is important that an individual risk assessment is performed for each user on a case-by-case basis. This is required to assess both the suitability of the grab handle for a particular client and the compatibility between the grab handle and the bed it is being fitted to.

WARNING

Ensure the grab handle is suitable for the bed it is being fitted to.

Ensure the grab handle is in the correct position. Ensure the securing clamps are tight and are checked at regular intervals Do not allow the grab handle to cause an increased risk to the patient.

It is important that an individual risk assessment is performed for each user on a case-by-case basis. This is required to assess both the suitability of the grab handle for a particular client and the compatibility between the grab handle and the bed it is being fitted to.

Care must be taken with rising backrests, as the gap between the two components could become less than 25mm in certain situations. If the bed has removable mattress guides, it is recommended to remove the ones closest to the grab handle to prevent any collision between the grab handle and mattress guides.

The grab handle and all components should be stored in a warm dry environment when not in use.


If you have any doubt about the suitability or fitting of the grab handle, contact Harvest Healthcare Ltd.

GRAB HANDLE INSTALLATION

FITTING THE ULTRA LOW GRAB HANDLE


The following instructions show how to fit the **HLDWUL685** grab handle to a **Woburn Ultra Low Bed only**.

- 1 Ensure that the screw fixings are loosened off and the clamping brackets are at the highest position.
- 2 Ensure the **black spacers** are secure in place. (The black spacers can be removed to allow the grab handle to be used on other manufactures beds with different size tubes but you must carry out a risk assessment to ensure the two products are compatible first)
- 3 Fit the grab handle to the side of the sleeping platform **ensuring that it is either less than 60mm from the headboard or more than 318mm from the headboard**.

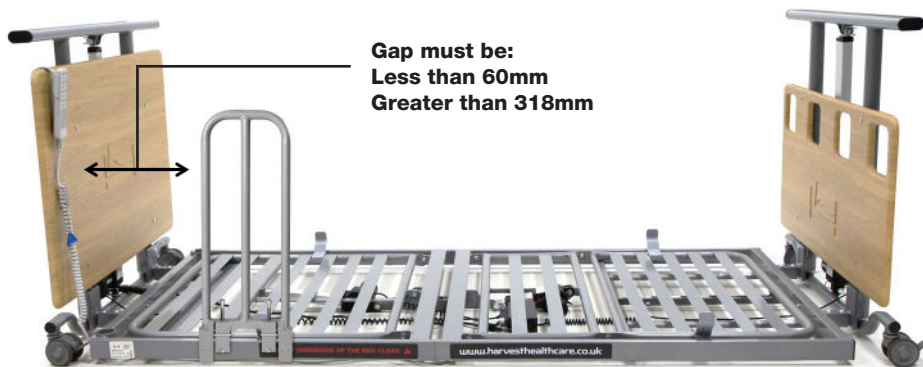

Please pay attention to the sleeping platform fixings as the grab handle will not fit over these fixings. It is recommended that the mattress guide by the grab handle is folded away or removed.

- 4 Tighten up the fixing bolt until the clamping brackets come into contact with the sleeping platform. Ensure the clamping brackets are in contact with the sleeping platform and **not** any of the mattress platform fixings.
- 5 Tighten the clamping levers ensuring that the grab handle is securely attached to the bed frame.

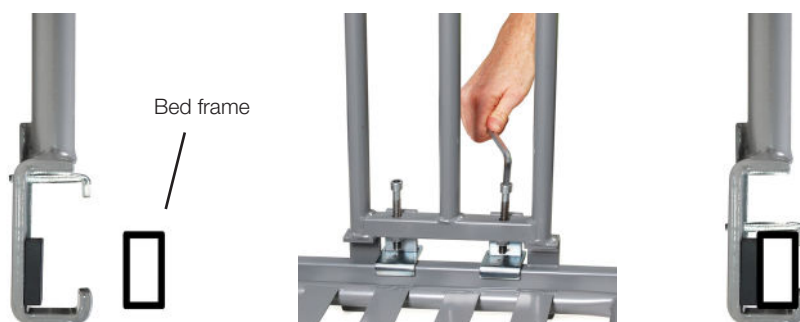

- 6 Secure fixing bolt using the allen key supplied. Cover the tops of the fixing bolts using the rubber covers.
- 7 Check the grab handle is securely fixed and operate the side rails to ensure there is no conflict between the two.
- 8 The grab handle and its fixings must be checked for any defects after each use and once again before each new installation, once installed the fixings should be checked for security at regular intervals.

