

CLEANPatch®

Mattress Repair Patch



Description:

- CleanPatch is a medically safe, proven effective patch for repairing hospital equipment with tears such as mattresses, inspection couches, plinths, wheelchair cushions and other PU covered or vinyl coated equipment.
- It is clinically validated by infection control professionals and will prolong the equipment life.
- Simply place over any tears or holes for an instant mend that will prolong equipment life, while maintaining infection control.

Feature:



- Clinically Tested
- Cost-savings by allowing repair rather than replace
- Safe and simple

Product Codes

PATCH11003	Small (2inch) Box of 10 + Healthcare Implementation Kit
PATCH10003	Medium (3.5x3.5inch) Box of 5 + Implementation Kit
PATCH12003	Large (3.5x6inch) Box of 5 + Implementation Kit
PATCH11001	Small (2inch) Box of 30 + Implementation Kit
PATCH10001	Medium (3.5x3.5inch) Box of 20 + Implementation Kit
PATCH12001	Large (3.5x6inch) Box of 10 + Implementation Kit
PATCH23001	Variety Kit - Box of 10 (x5 S, x3 M, x2 L) + Implementation Kit
PATCH21001	V Strip (1.75x4inch) - Box of 10 + Implementation Kit
PATCH22001	V Rectangle (5.75x8inch) - Box of 5 + Implementation Kit
PATCH23001	V Variety Kit - Box of 10 (7 Strip + Rectangle) + Implementation Kit
PATCH24001	V Starter Kit - Box of 3 (2 strip + 1 Rectangle) + Implementation Kit
PATCH31001	CleanPatch - P - Box of 10

Using the Patch:

1. Clean and dry the mattress. You can use standard mattress cleaning chemicals, such as sodium hypochlorite.
2. Remove white bottom
3. Apply patch from the center
4. Press firmly and smooth down the patch
5. Remove clear top
6. Clean patch and area

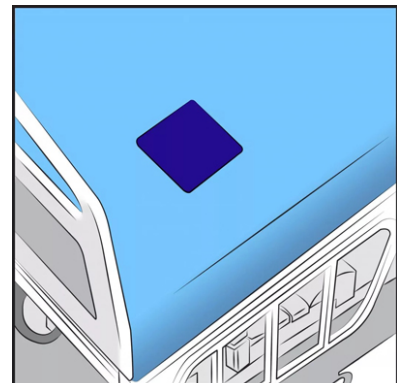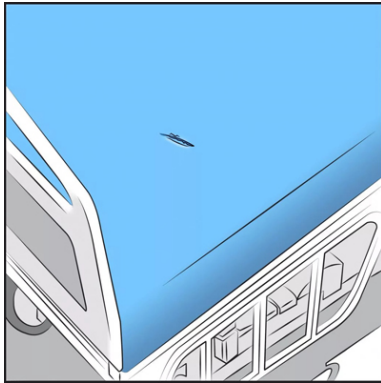

For more information see the videos on our website:
<https://harvesthealthcare.co.uk/product/cleanpatch/>

Supplier Contact Details:

t: +44(0)1709 377172 e: enquiries@harvesthealthcare.co.uk; w: www.harvesthealthcare.co.uk
Sheaf house, Bradmarsh Way, Bradmarsh Business Park, Rotherham UK, S60 1BW

Important Safety Notice

Harvest Healthcare Ltd supplies equipment and appliances to cover a range of medical conditions. Specific choice of equipment is the sole responsibility of the clinical practitioner or carer overseeing individual patient care. Harvest Healthcare Ltd supplies a range of products such as foam mattresses, active air mattresses, fall protection mats and bed rail protection bumpers. These products are usually covered in vapour permeable PU fabric that could cause risk of suffocation in the event of prolonged facial contact.
(Mattress risk categories are to be used as a guide only) Risk assessments specific to user must be carried out prior to the use of all products.
Products and data continually undergo development and may therefore deviate from the details given.