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




General User/Safety Guide

# STATIC MATTRESS LENGTH EXTENSION INFILL WITH STRAPS





# CONTENTS

**WARNINGS & CAUTIONS**

**PRODUCT OVERVIEW**

**GENERAL INFORMATION**

**INSTALLATION**

**BUMPER EXTENSION INSTALLATION**

**CLEANING AND CARE**

# WARNINGS & CAUTIONS



**READ THIS INSTRUCTION MANUAL CAREFULLY. PLEASE NOTE IN PARTICULAR THAT THE SAFETY INSTRUCTIONS MUST BE OBSERVED.**



## **WARNING**

Do not allow sharp objects to puncture the cushion material.

The Cushion and should be cleaned between patient uses.

Do not use bleach, chlorine releasing agents in concentrations over 1000 ppm, solvents or alcohol-based cleansers, e.g. Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline as these will destroy the cushion material.

Suitable for continuous use.

Do not modify the cushion in any way.

Not for use in an outdoor environment.

Store the cushion in a clean and dry environment, out of direct sunlight.

Do not use this cushion for lifting the patient. This will damage the cushion and could put the patient at risk.

This product is fire rated. The cushion material is tested to BS7175:1989 Crib 5. Use of this product should be subject to a risk assessment in which all hazards are considered.

### **SYMBOLS USED**



Wash at 80°C (cover only - remove foam)



Tumble dry on Low



Do Not Dry Clean



Do Not Bleach



Do Not Iron



CE Mark



UKCA Mark



Do Not Use Phenols



No Smoking



This symbol indicates general hazards. There is a danger to life and health.

# PRODUCT OVERVIEW

## PRODUCT DESCRIPTION

### Static Mattress Length Extension Infill with Straps

The Static Mattress Length Extension Infill with straps is designed to allow it to be used and attached to the backrest of the mattress platform section to keep it in place as the platform is profiled. Usually required when extending a bed length.



# GENERAL INFORMATION

## INSTRUCTIONS FOR USE

Check the product for damage, and if it is the correct type of product for the bed, before carrying out the installation.

- If the cover, webbing or any other part of the product looks damaged do not fit it to the bed.
- Refer to the product codes and images to see the fitted product.
- Before installing this product read the instruction for use thoroughly and seek advice if you are unsure how to fit the product.

To help prevent injury it is important that patients and equipment are monitored regularly.

The Mattress infill may be delivered in a plastic cover for protection; these covers must be removed and disposed of before the equipment can be used.

Please dispose of the plastic as specified by your local waste disposal service authority.

## INTENDED PURPOSE

The Static Mattress Length Extension Infill with straps is designed to allow it to be used and attached to the backrest of the mattress platform section, keeping it in place as the platform is profiled. This is usually required when extending the length of a bed.

## INSPECTION RECOMMENDATIONS

The Static Mattress Length Extension Infill should be checked regularly to ensure it remains 'fit for purpose', effective and pose no risk of infection to either the patient or the carer.

Harvest Healthcare recommend that a thorough inspection of both the exterior (cover) zips, straps and buckles is carried out weekly, or each time a new patient is introduced to this equipment. Visual checks should be carried out daily to identify any significant signs of damage or infection risk.

If any signs of contamination are identified, the bumper system should be withdrawn from use immediately and cleaned.

## CLEANING AND CARE

The Static Mattress Length Extension Infill has protective PU cover. The cover should be cleaned regularly in accordance with your organisation's disinfection protocol. (See page 10)



**Facial contact with any covering material such as that used on this product can cause suffocation and a Risk Assessment should be carried out prior to the use of this product.**

# GENERAL INFORMATION

## **MOVING, HANDLING AND STORAGE**

Where possible, the Mattress Length Extension Infill should be stored flat to prevent damage to the product.

The Mattress Length Extension Infill is to be stored between -5 to +40 degrees, in a dry environment. Do not expose to high humidity or heat.

**Do not drag or pull the Mattress Length Extension Infill by the cover.**

## **SAFE USE**

This product is intended as an aid to prevent accidental injury. With a carer present check the material, zips, webbing, and buckles to ensure that these parts of the Static Mattress Length Extension Infills have not been damaged and do not introduce gaps or a hazard that could be increased the risk of entrapment suffocation or injury to the end user or carer.

Before and during use of the Static Mattress Length Extension Infill, consider the effectiveness of the system, including if the Mattress Length Extension Infill is the correct size and type for the mattress.

Seek advice to ensure the correct Mattress Length Extension Infill is selected before installation as per the manufactures instructions.

## **WARRANTY**

All Harvest Healthcare Static Mattress Length Extension Infills are covered by warranty for a period of 1 year. Damage through incorrect use will invalidate this warranty.

## **GUARANTEE**

Harvest Healthcare guarantees to repair or replace all goods issued to its customers which are found to be defective whilst still under their applicable warranty period. All warranties are subject to our standard terms and conditions of sale which can be found on our website or by request to Harvest Healthcare Ltd.

# INSTALLATION

1. Raise the backrest section (to give access to the underside)
2. Place the mattress length extension infill on the backrest as shown with the hook and loop section through the backrest.



3. Lay the mattress length extension infill down and feed the hook and loop section through the buckle and secure (as shown below)





# INSTALLATION

4. Repeat the process for the second strap.
5. Ensure the Mattress Length Extension Infill is secured in place and the Text saying TOP SIDE UP is upward
6. The Mattress Length Extension Infill should look like this.



# CLEANING & CARE

## **WARNING**

Eye protection, gloves, and protective clothing should be worn when carrying out cleaning and disinfection procedures.

When disinfecting the system, Harvest Healthcare recommends the following guidelines which have been developed to comply with recognised infection control procedures.

These procedures are also to be used to prevent cross infection when transferring the system between patients.

### **STATIC MATTRESS LENGTH EXTENSION INFILL COVER**

During general use, the Mattress Length Extension Infill cover can be cleaned by wiping with a mild detergent solution.

Where necessary the cover can be laundered or wiped clean. Where there is staining or body fluids on the outer cover, wash thoroughly with soap and water, then wipe with a sodium hypochlorite solution diluted to 1000ppm before laundering. Covers may be laundered if the foam can be removed.

### **INSTRUCTIONS ON HOW TO LAUNDRY THE OUTER COVER**

Lay the Mattress Length Extension Infill cover down on a flat surface and unzip the pocket to gain access to the internal foam padding. Remove the foam padding.

The Mattress Length Extension Infill can now be laundered.

- 1 Pre wash cold 10 minutes
- 2 Main wash 80°C 10 minutes
- 3 Followed by cold rinses and extraction.

Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.

Tumble dry at low heat.

Do not iron.

Ensure that the cover is thoroughly dried before inserting the foam and placing the Mattress Length Extension Infill into use or storage.



### **HYPERCARBONATE AND PHENOL-BASED SOLUTIONS SHOULD NOT BE USED AS THESE WILL DAMAGE THE PRODUCT COVER**

### **DISPOSAL**

If contaminated, please dispose of as clinical waste.

If not contaminated dispose of as household waste at your local approved facility.

### **Note:**

- Carry out a risk assessment to ensure the bumper is suitable for the users' requirements.
- In the event of the Handset lock pocket becoming damaged discontinue use immediately and contact Harvest Healthcare for advice.
- You will need a small padlock to secure the handset in the Lock Pocket.
- If you have any questions, please contact Harvest Healthcare before use.





harvesthealthcare®  
a prismhealthcare company


Sheaf House, Bradmarsh Business Park,  
Bradmarsh Way, Rotherham S60 1BW

T +44 (0)1709 377172

F +44 (0)1709 377173

E [sales@harvesthealthcare.co.uk](mailto:sales@harvesthealthcare.co.uk)

[www.harvesthealthcare.co.uk](http://www.harvesthealthcare.co.uk)

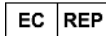

European Healthcare & Device Solutions (Ireland) Ltd.,  
Stratton House, Bishopstown Road, Cork T12 Y9TC, Ireland.