



harvesthealthcare®
a prismhealthcare company

General User/Safety Guide

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
GRAB HANDLE SAFETY SLEEVE

BED ACCESSORIES



www.harvesthealthcare.co.uk

CE
UK
CA



CONTENTS

WARNINGS & CAUTIONS

PRODUCT OVERVIEW

GENERAL INFORMATION

INSTALLATION

BUMPER EXTENSION INSTALLATION

CLEANING AND CARE

WARNINGS & CAUTIONS



READ THIS INSTRUCTION MANUAL CAREFULLY.



PLEASE NOTE IN PARTICULAR THAT ALL THE SAFETY INSTRUCTIONS MUST BE OBSERVED.

WARNING

The Grab Handle Safety sleeve is only for use on the Harvest Healthcare range of grab handles. Before fitting the Grab handle Safety sleeve, it is important that an individual risk assessment is performed for each user on a case-by-case basis. This is required to assess both the suitability of the Grab handle Safety sleeve for a particular client and the compatibility between the Grab handle Safety sleeve and the grab handle it is being fitted to as well as other accessories/ equipment being used.

The Grab handle Safety sleeve and its fixings must be checked for any defects after each use and once again before each new installation, once installed the fixings should be checked for security at regular intervals.

CAUTION

Ensure the Grab handle Safety sleeve is suitable for the grab handle it is being fitted to and installed in the correct position.

Ensure the securing fixings are secure and are checked at regular intervals. Do not allow the Grab handle Safety sleeve pocket to cause an increased risk to the patient.

The Grab handle Safety sleeve should be stored in a warm dry environment when not in use. If you have any doubt about the suitability or fitting of the Grab Handle Safety Sleeve, contact Harvest Healthcare Ltd. Report any issues involving the use of Lock pocket to Harvest Healthcare Ltd.

GENERAL INFORMATION

EXPLANATION OF THE SYMBOLS USED IN THE IFU AND THE LABELS.



Read instructions / consult manufacturers guide.



This symbol indicates general hazards. There is danger to life and health.



Conformity mark in accordance with the Medical Devices Regulation (EU) 2017/745.



Declaration of Conformity for the UK



Fragile, handle with care.



This way up.



Keep dry.



Recycling symbol. Refers to packaging that can be recycled (cardboard).



Operation and storage temperature.



UK Representative.



Distributor



Maximum permissible load.



Manufacturer of the medical device



Manufacturing date



Medical Device



Wash at 80°C

GENERAL INFORMATION



Hand wash



Do not bleach



Do not dry clean



Do not iron



Tumble dry on low

PRODUCT OVERVIEW

PRODUCT DESCRIPTION

Grab Handle Safety Sleeve **Product Code: HLBGHS**

The Grab Handle Safety Sleeve is designed to work with the Harvest range of grab handles. Its purpose is to ensure patient safety and reduce the risk of entrapment when using grab handles. This will help mitigate any potential risks associated with the use of grab handles and beds and ensure a safer environment for the patient.

It works with Standard Grab Handle (HLB685), Ultimate Grab Handle (HLB686) and Ultra Low Grab Handle (HLDWUL685).



GENERAL INFORMATION

INSTRUCTIONS FOR USE

Check the product for damage and ensure it is the correct type of product for the bed before carrying out the installation.

- If the netting or any other part of the product looks damaged, do not fit the Grab Handle Safety Sleeve.
- Refer to the product code and pictures to make sure you're using the right instructions for use.
- Before installing this product, read the instructions for use thoroughly and seek advice if you are unsure how to fit the product.

To help prevent injury, it is important that patients and equipment are monitored regularly. The Grab Handle Safety Sleeve may be delivered in a plastic cover for protection; these covers must be removed and disposed of before the equipment can be used. Please dispose of the plastic as specified by your local waste disposal service authority.

INTENDED PURPOSE

The Grab Handle Safety Sleeve is designed to work with the Harvest range of grab handles. Its purpose is to ensure patient safety and reduce the risk of entrapment when using grab handles. This will help mitigate any potential risks associated with the use of grab handles on the bed and ensure a safer environment for the patient.

INSPECTION RECOMMENDATIONS

The Grab Handle Safety Sleeve and any equipment it is attached to should be checked regularly to ensure it remains 'fit for purpose', effective, and poses no risk of infection to either the patient or the carer.

Harvest Healthcare Ltd recommends that a thorough inspection of the zips, hook and loop, and mesh be carried out weekly, and each time a new patient is introduced to this equipment. Visual checks should be carried out daily to identify any significant signs of damage or infection risk.

If any signs of contamination are identified, the system should be withdrawn from use immediately and cleaned.

CLEANING AND CARE

The Grab Handle Safety Sleeve has a mesh fabric with 2 zips and hook and loop. The Grab Handle Safety Sleeve should be cleaned regularly in accordance with your organisation's disinfection protocol (see page 12).

GENERAL INFORMATION

MOVING, HANDLING AND STORAGE

Where possible, the Grab Handle Safety Sleeve should be stored flat to prevent damage to the product. The Grab Handle Safety Sleeve is to be stored between -5 to +40 degrees in a dry environment. Do not expose it to high humidity or heat.

SAFE USE

This product is intended as an aid to prevent accidental injury. With a carer present, check the mesh material, zips, and hook and loop to ensure these parts of the Grab Handle Safety Sleeve or any product it is attached to have not been damaged.

Before and during use of the Grab Handle Safety Sleeve, consider the effectiveness of the system, including any product the Grab Handle Safety Sleeve has been attached to. The hazard of entrapment can occur if the system has not been fitted correctly. Seek advice to ensure the correct product is selected before installation as per the manufacturer's instructions.

The grab handle has been tested for a maximum patient weight of 60 kg. However, the covering sleeve is not weight-bearing; it is designed to help prevent accidental injury.

WARRANTY

BUMPER SYSTEMS


The Harvest Healthcare Grab Handle Safety Sleeve is covered by a warranty for a period of 1 year. Damage through incorrect use will invalidate this warranty.

GUARANTEE



Harvest Healthcare Ltd guarantees to repair or replace all goods issued to its customers that are found to be defective while still under their applicable warranty period. All warranties are subject to our standard terms and conditions of sale, which can be found on our website or by request from Harvest Healthcare Ltd.

INSTALLATION



1. Ensure the Grab Handle is manufactured by Harvest healthcare, is installed correctly and in the correct position.
2. Present the Grab handle safety sleeve to the grab handle below ensuring the zips start at the top and the hook and loop is at the bottom.

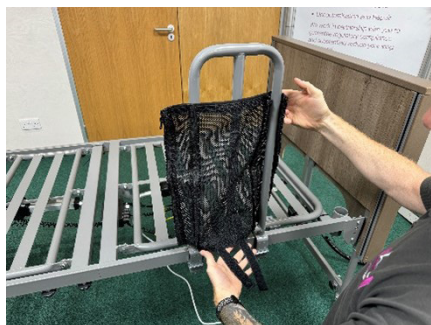

3. Connect the first zip.

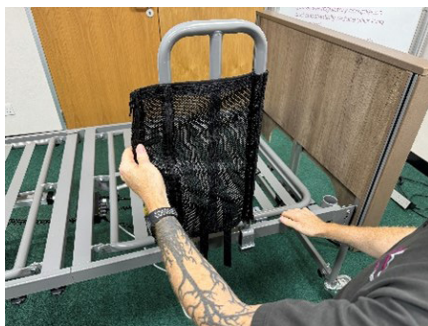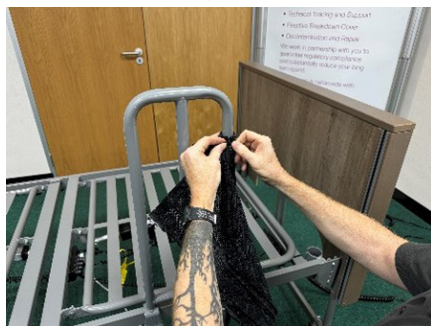

4. Connect the second zip ensuring the central tube of the grab handle is accessible to the person inside the bed.

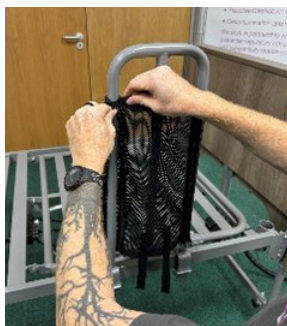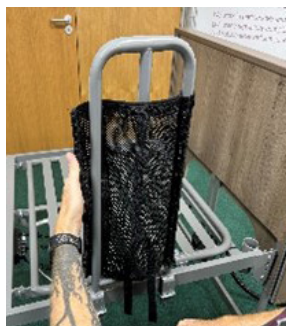

INSTALLATION

5. Push the 2 hook and loop straps towards the inside of the bed and around the lower tube of the grab handle then connect the hook with the loop on the outside of the mesh.



6. Once installed the system should look like this from the inside of the bed.



After Installation Re-Check

- The grab handle is securely fitted to the bed and in the correct position (see grab handle IFU).
- Both zips are fully closed and on the outside of the grab handle, away from the person in the bed.
- The hook and loop connection is secure and on the outside of the grab handle, away from the person in the bed.
- The central tube of the grab handle is accessible to the person in the bed.
- There is no risk to the person in the bed from the grab handle or the Grab Handle Safety Sleeve.

CLEANING & CARE



WARNING

Eye protection, gloves, and protective clothing should be worn when carrying out cleaning and disinfection procedures.

When disinfecting the system, Harvest Healthcare recommends the following guidelines which have been developed to comply with recognised infection control procedures.

These procedures are also to be used to prevent cross infection when transferring the system between patients.

Where necessary the Grab handle safety sleeve can be wiped clean. Where there is staining or body fluids on the outer cover, wipe thoroughly with soap and water, then wipe with a sodium hypochlorite solution diluted to 1000ppm before laundering.

Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these could damage the Handset lock pocket.

Ensure that the Grab handle safety sleeve is thoroughly dried before placing it into use or storage.



HYPERCARBONATE AND PHENOL-BASED SOLUTIONS SHOULD NOT BE USED AS THESE WILL DAMAGE THE PRODUCT COVER

DISPOSAL

If contaminated, please dispose of as clinical waste.

If not contaminated dispose of as household waste at your local approved facility.

Note:

- Carry out a risk assessment to ensure the Grab Handle Safety Sleeve is suitable for the users' requirements.
- The Grab Handle Safety Sleeve is only recommended for use on the Harvest Healthcare range of grab handles.
- In the event of the Grab Handle Safety Sleeve becoming damaged discontinue use immediately and contact Harvest Healthcare for advice.
- If you have any questions, please contact Harvest Healthcare before use.

Scan the QR code below to watch the Grab Handle Safety Sleeve installation video.





harvesthealthcare®
a prismhealthcare company

Sheaf House, Bradmarsh Business Park,
Bradmarsh Way, Rotherham S60 1BW



T +44 (0)1709 377172

F +44 (0)1709 377173

E sales@harvesthealthcare.co.uk

www.harvesthealthcare.co.uk



European Healthcare & Device Solutions (Ireland) Ltd.
Stratton House
Bishopstown Road
Cork T12 Y9TC, Ireland